The Sea-Band operates by exerting pressure on the acupressure point (called the Nei-Kuan Point) on each wrist. It takes only a few minutes to become effective.

The Sea-Band provides effective relief from all forms of Travel Sickness without causing drowsiness or other side effects.

It has also been found to be effective in other conditions that produce nausea, such as pregnancy, anaesthesia and chemotherapy.

The Sea-Band may be used repeatedly or worn continuously and can be hand washed using soap or mild detergent. Remove the bands if any swelling occurs.

Sea-Band is suitable for adults and children.

HOW TO POSITION THE SEA-BANDS CORRECTLY

1 Place your middle three fingers on the inside of your wrist with the edge of the third finger on the wrist crease. The Nei-Kuan point is just under the edge of your index finger between the two central tendons.

2 Position the button facing downward over the Nei-Kuan point. Repeat the process for the other wrist.

One band must be worn on each wrist to be effective. Sea-Bands can be put onto your wrist before you start travelling, or after you have started your journey.

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Sea-Band is a registered trademark